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Root canal instruments · Instruments pour le traitement des canaux radiculaires · Instrumentos para conductos radiculares

### ACURATA DISCOVERY ENDOBOHRER – EXTRALANG

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### ACURATA DISCOVERY ENDO-BUR – EXTRA LONG

- long slender neck
- extra long version, all-over length 34 mm
- to expand the root canal entrance
- for the Isthmus-preparation
- lasermarking 6 mm, 9 mm and 12 mm for the depth estimate
- color-coded for a secure recognition of the sizes, individual processing
- one-piece tungsten carbide construction – for an optimal stability
- max. speed: 20.000 min<sup>-1</sup>, opt speed: 1.500 – 2.000 min<sup>-1</sup>

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Longneck-Endo-Bur for the Isthmus-preparation.

- extra long and slender neck
- good sight on the working surface, safe in application
- sharp tooththing
- clear identification by the green colour ring

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**Recommendations:** Optimum Speed: 1.000–2.000 min<sup>-1</sup>, with light contact pressure. Do not lever or wedge the instrument.

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#### Recommendations for Use and Safe Operation of Root Canal Instruments:

- K-Reamer: push and rotary motion clockwise up to max. 90°
- K-File: filing motion clockwise up to max. 45°. Hedstroem: push and pull (with or without rotary) motion up to max. 90°
- Usage in ascending order of size (do not skip any size)
- high flexible Nickel-Titanium instruments are especially for heavy curved root canals. The file has not to be bent before use
- protect the instruments against aspiration (Rubber dam – Safety chain – Safety cord)
- clean the instruments properly before sterilization
- Sterilization in autoclave may corrode steel instruments
- do not use bent or damaged instruments

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Color of stops indicates the length of working part

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